

Padded Malleable Pet Splints

User Instructions



Choose the appropriate size for the limb area to be supported. The splint should cover just protruding beyond toes on distal /proximal end- front leg above elbow, rear leg above hock.

Bend the splint to comfortably fit the contour of the limb.

Apply padding (cast padding or gauze) between skin surface and splint for added support and comfort.

Place the bended splint on the area to be supported.

Secure the splint in place with cohesive bandage.

Cat. No	Name	Length cm	Width cm
1650F	Padded Pet Splint XS, 5/pk	18,4	6,7
1650aF	Padded Pet Splint S, 5/pk	22,9	7,6
1650bF	Padded Pet Splint M, 5/pk	26,7	10,2
1650cF	Padded Pet Splint L, 5/pk	31,8	10,2
1650dF	Padded Pet Splint XL, 5/pk	34,9	10,2