GB • User instructions

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Description

Armlength examination glove for veterinary use only.

Generally

Before use, it should be tested / ensured that the glove fit the user with the appropriate size to achieve the best possible comfort and safety at work.

Category

The glove is a category I product in accordance with the PPE Regulation (EU) 2016/425 for personal protective equipment.

Test data

The gloves are tested with leak test according to AQL 2.5 inspection level S1 / S3.

Application / qualities / use

For doing rectal and vaginal exploration and procedures on horses and farm animals.

The purpose of the gloves is to avoid the person performing these procedures to get in skin contact with feces and body fluids from the animals examined.

The gloves should be sufficiently lubricated when performing these procedures to avoid them to tear from the impact of the procedures.

The gloves are for single use only and the gloves should be replaced with new ones for any prolonged procedures.

Treatment / storage

Must be stored in a dark, dry and cool room in the original packaging. Avoid storage in direct sunlight.

Transportation by the vet must be inside the dispenser box.

Material

The gloves are made of polyethylene and material is formulated and manufactured in accordance with the requirements of food contact recommendations or regulations.

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