GB • User instructions

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Description

Gown for veterinarians during veterinarian surgery. For veterinary use only.

Generally

Before use, it should be ensured that the correct size is used to achieve the best possible comfort and safety at work.

Category

The colic gown is a category I product in accordance with the PPE Regulation (EU) 2016/425 for personal protective equipment and meets the essential requirement in Appendix II of Regulation (EU) 2016/425 of 9 March 2016 on personal protective equipment.

Application / qualities / use

The colic gowns are waterproof and have internal protection on front and sleeves, knitted cuffs and stitching with overlocking machine. Double internal belts; one-piece body. Crew neck with strap; adjustable collar closure with Velcro, 4 waist belts for internal / external closure superimposed. Equipped with 2 hand towels and reverse folding for programmed opening and wrapped in medical paper.

The product is sterilized and individually packaged.

If worn correctly, the coat should be comfortable, without preventing or limiting any movement.

How to take on the gown

- Gowns should be taken on with the help of a second person
- Take the coat out of the box with clean, dry hands or clean gloves
- Spread the coat on the medical paper from the packing
- Slide hands and forearms inside the sleeves, keeping the arms at shoulder height and far from the body
- Tuck your arms completely by pulling your hands out of the cuffs
- Assistant closes coat behind back and knots laces.

Treatment / storage

Must be stored in a dark, dry and cool room in the original packaging. Avoid storage in direct sunlight. Transportation by the vet must be inside the dispenser box.

Material

The colic gowns are in waterproof TNT SMMS, 35GSM and PP+ PE (55GSM), blue colour.

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