Instruction for Use

Insertion and handling FioniaVet Foley Urinary Catheter with stilet

Italiano: [vedere pagina 2](#)

English

This instruction is valid for the products listed below. Please read the document and information on packing before use.

Item No	Item name	Size
561418 - 561428	FIONIAVET Silicone Foley Catheter, 2 Way, w/stilet, 10/pk	All sizes

- Prepare procedure according to local procedures.
- Create a sterile field for next step e.g. drape with a hole, open the peel-pack and place the sterile catheter on the sterile drape.
- Change to sterile gloves.
- Remove/rip of the back and the front end of the inner packaging and hold the catheter straight
- **IMPORTANT: loosen the stilet just a tiny bit to avoid it gets stuck during removal. Be aware of the fact, that the smaller size and the longer the catheter is, the more important this paragraph is.**
- Apply sterile lubricant to the tip of the catheter.
- Use e.g. the middle part of the inner packaging as handle and insert the catheter slowly into the urethral opening.
- When urine starts to flow, insert the catheter even further to assure the balloon is inside the bladder. Attach catheter e.g. to urine bag or stop flow by spigot.
- Hold the catheter as straight as possible according to the animal's anatomy and draw back the stilet **very gently** approx. 10 cm and fill the balloon with the recommended amount of saline water printed on the colored hub of the catheter,
- Draw back very gently the rest of the stilet.
- Withdraw also gently the catheter back until resistance of the inflated balloon.

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